



**tag**  
DENTAL

BY NOGA  
MEDICAL

# Internal Hex.

**The Dental Division: Offers superior quality implantology products:  
Dental implants, Prosthetics, Surgical instruments, and biological materials.**

T.A.G. is ISO 13485:2016 certified including MDSAP by GMED.

All products are carry the CE Mark according to Annex II (Notified Body – GMED)

The scope of certifications covers the design, development, manufacturing and marketing of Dental Implants, Abutments and Accessories.

The TAG Dental implant system products are cleared for marketing in the USA\* (K143326) and Canada - not all products are available in all markets.


Our expert development team with its many years of dental experience is pleased to introduce a new generation of implants based on platform switching and reverse buttress thread pattern:

All implants are manufactured from biocompatible Titanium alloy Grade 23 (Ti 6AL 4V ELI) .The initial stability of the implant is a critical factor in determining the final osseointegration results. Titanium alloy designed for all bone types.

# Index

---

|                   |    |
|-------------------|----|
| Implants .....    | 5  |
| Axis .....        | 11 |
| Massif .....      | 23 |
| Crestone .....    | 31 |
| Restoration ..... | 37 |
| Instruments ..... | 55 |
| Sybone .....      | 75 |
| Sycure .....      | 79 |
| Digital .....     | 85 |

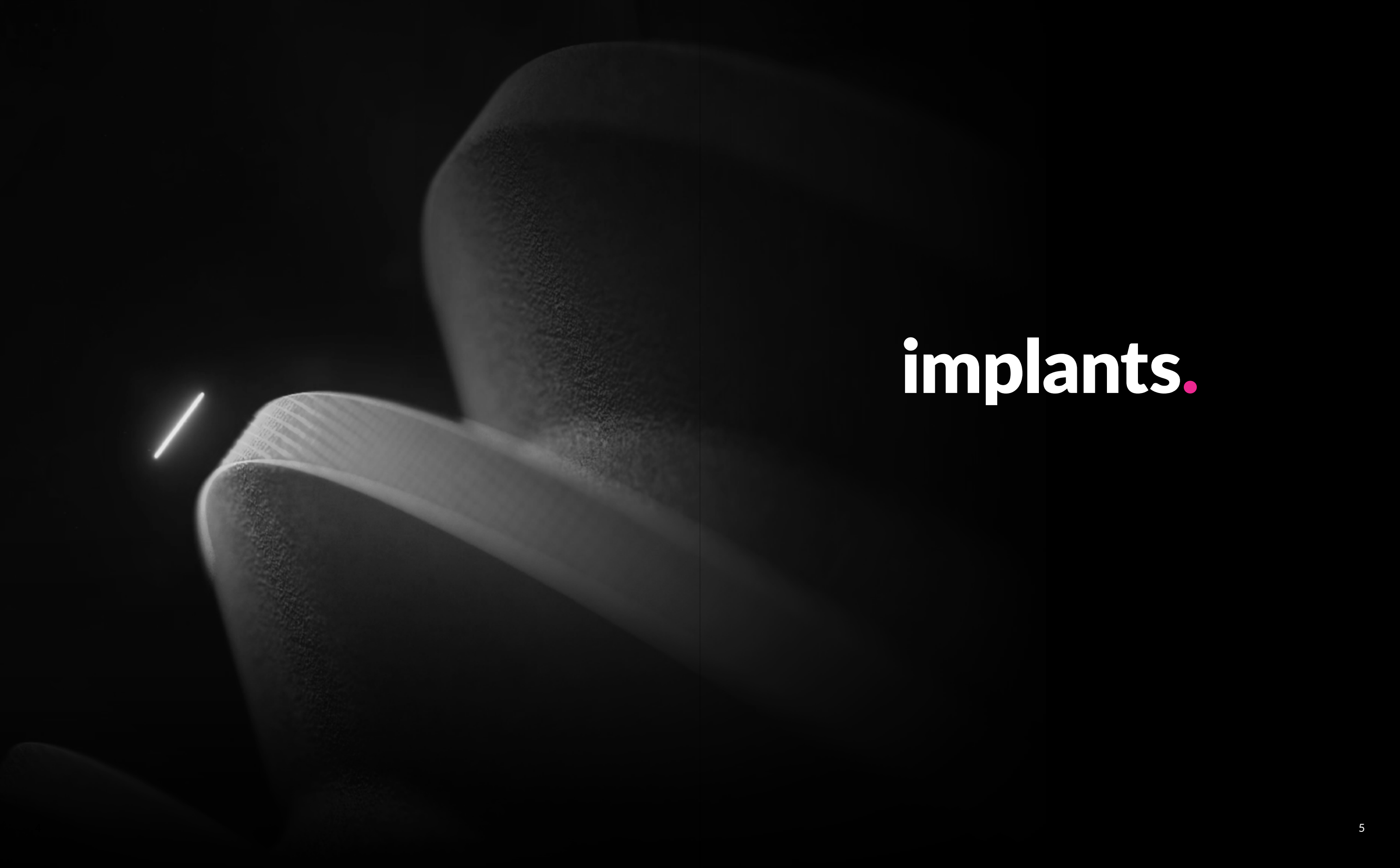
The background is a dark, moody composition. On the left, a bright, thin white line glows diagonally. To its right, a curved, textured surface, possibly a piece of fabric or a medical device, is partially illuminated, showing fine details and a soft gradient. The rest of the image is in deep shadow.

**implants.**




# Advantages

## ONE IMPLANT SYSTEM

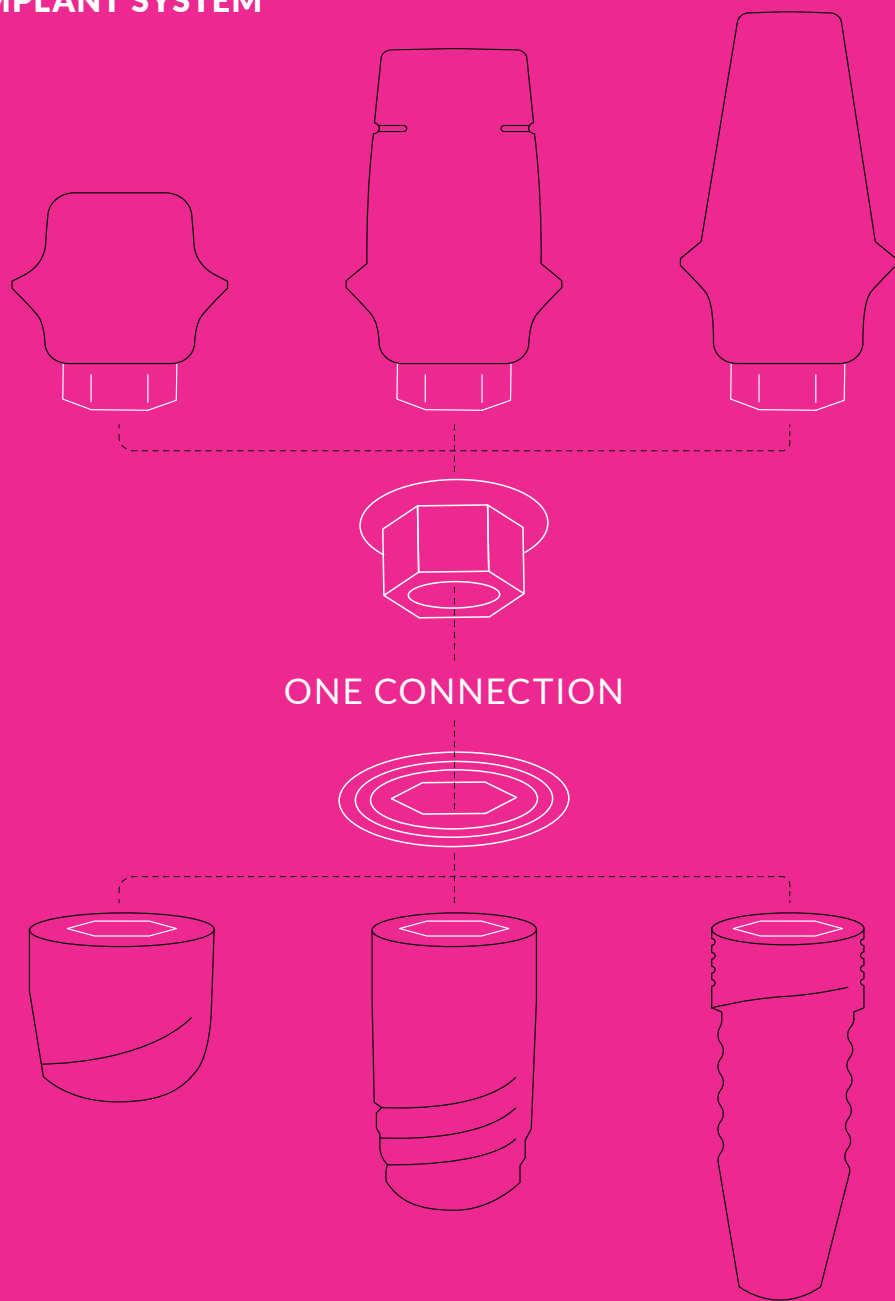


PLATFORM SWITCHING

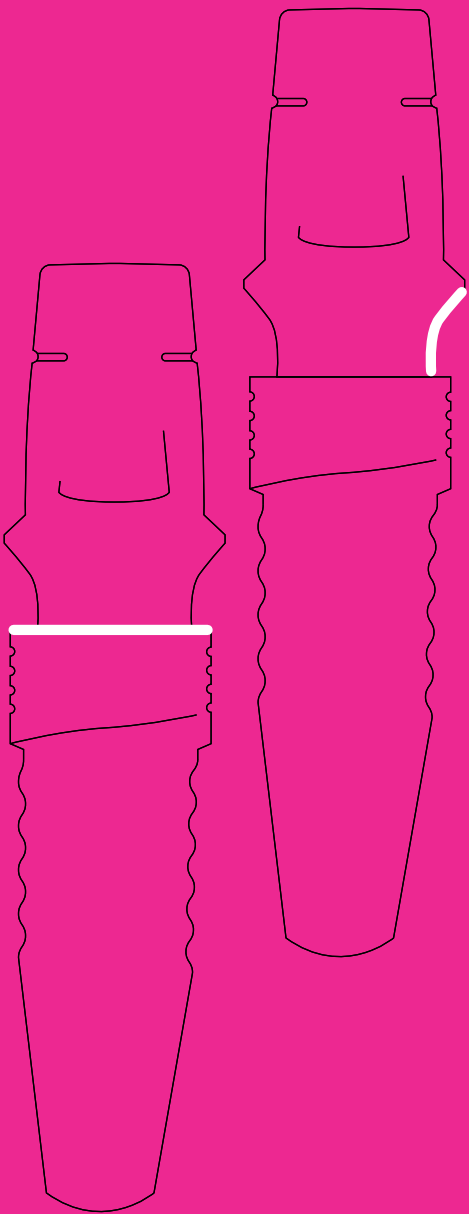


REVERSE CROWN  
CONCAVE PROFILE


# Macro Porous

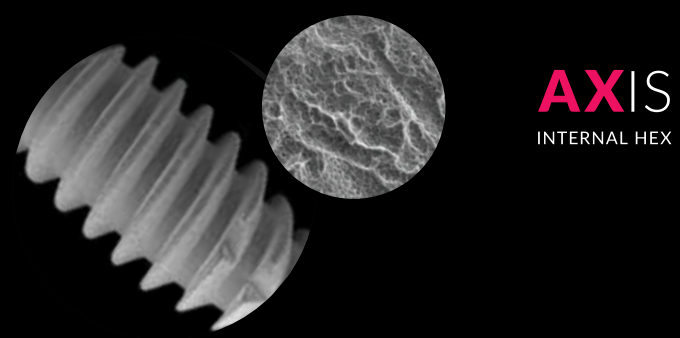

**AXIS**  
INTERNAL HEX

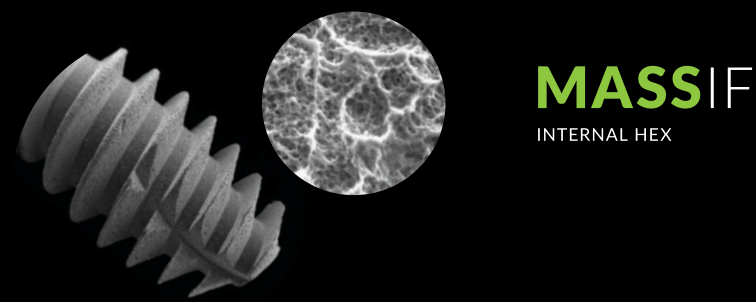

**MASSIF**  
INTERNAL HEX

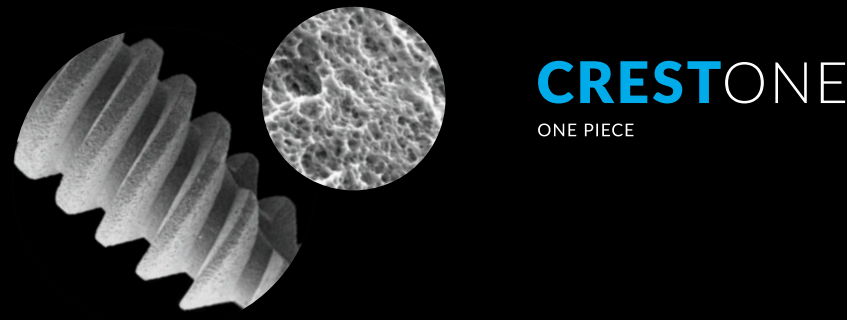

**CRESTONE**  
ONE PIECE

| Thread Area | Flat  | Top   |
|-------------|-------|-------|
| C           | 29.57 | 27.86 |
| Ti          | 15.81 | 16.32 |
| O           | 52.58 | 53.54 |
| N           | 0.21  | 0.84  |
| Ca          | 0.24  | 0.31  |
| Si          |       |       |
| S           | 0.08  |       |
| V           | 0.29  | 0.54  |
| Cl          |       |       |
| P           |       |       |
| K           |       |       |
| Al          | 0.84  | 0.63  |
| Na          | 0.34  |       |
| Mg          |       |       |
| Zn          |       |       |



**AXIS**  
INTERNAL HEX

# AXIS Advantages



## ATTACHMENT

Internal hexagon 2.44mm.  
Provides antirotation.  
Provides positioning indication.  
Provides accurate and firm insertion using dedicated tooling.

## TOP

4 Micro threads at the implant crest module decreases stress and increases surface contact area in the cortical bone.

## GEOMETRY

Wide pitch progressive thread enables bone compression during insertion, ultimately improving bone volume support.

## SURFACE

The micro surface topography is achieved by blast technology followed by acid etching


## APICAL

Self-tapping allows exceptional cutting capabilities, and is responsible for increased stability during insertion as the implant carves, fills, and compresses bone.

# AXIS Implant Range

| Length (mm) | 6                                                                                     | 8                                                                                     | 10                                                                                    | 11.5                                                                                  | 13                                                                                    | 16                                                                                    |
|-------------|---------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|
| Ø 3.3       |                                                                                       |                                                                                       |    |    |    |    |
|             |                                                                                       |                                                                                       | IA1-33010                                                                             | IA1-33011                                                                             | IA1-33013                                                                             | IA1-33016                                                                             |
| Ø 3.75      |                                                                                       |    |    |    |    |    |
|             |                                                                                       | IA1-37508                                                                             | IA1-37510                                                                             | IA1-37511                                                                             | IA1-37513                                                                             | IA1-37516                                                                             |
| Ø 4.2       |  |  |  |  |  |  |
|             | IA1-42006                                                                             | IA1-42008                                                                             | IA1-42010                                                                             | IA1-42011                                                                             | IA1-42013                                                                             | IA1-42016                                                                             |
| Ø 5.0       |  |  |  |  |  |                                                                                       |
|             | IA1-50006                                                                             | IA1-50008                                                                             | IA1-50010                                                                             | IA1-50011                                                                             | IA1-50013                                                                             |                                                                                       |
| Ø 6.0       |  |  |  |  |  |                                                                                       |
|             | IA1-60006                                                                             | IA1-60008                                                                             | IA1-60010                                                                             | IA1-60011                                                                             | IA1-60013                                                                             |                                                                                       |

# One Connection Fits All

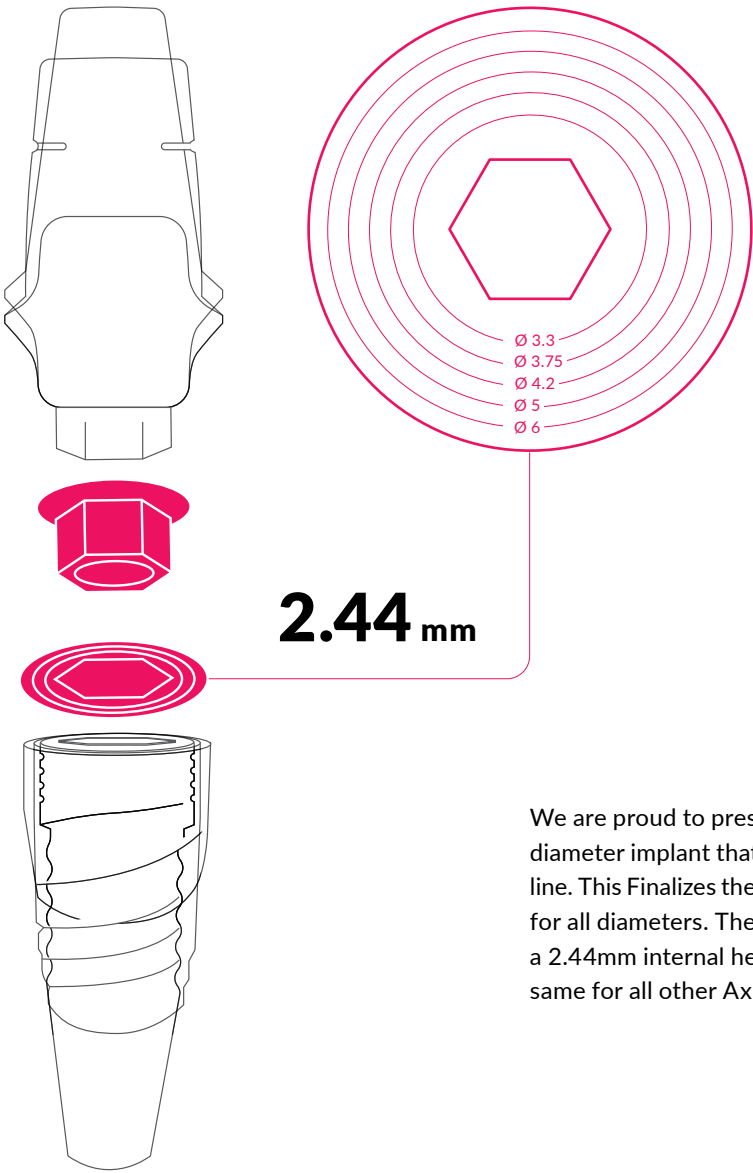

We are proud to present our new Axis 3.3mm diameter implant that completes the TAG Implant line. This Finalizes the concept of same connection for all diameters. The 3.3mm narrow implant has a 2.44mm internal hex connection, which is the same for all other Axis implants diameters.



**Due to the implant's special design:**

- Average static load test showed an average of 666N.
- Fatigue limit of implants tested obtained 340N with 5 million cycles.


Static and fatigue test performed on our 3.3mm implants at the Technion Institute (Haifa, Israel) demonstrated exceptional resistance to load and fracture compared to other implants.

**IMPLANT 3.3**

Static Loading Curve

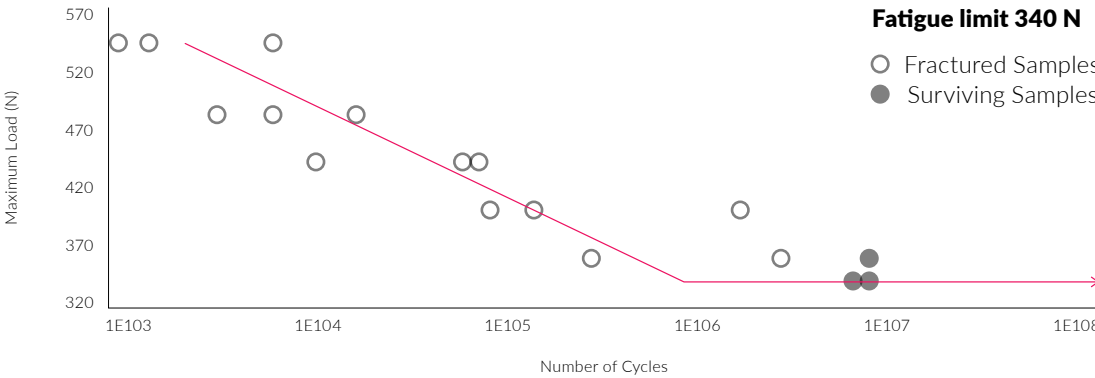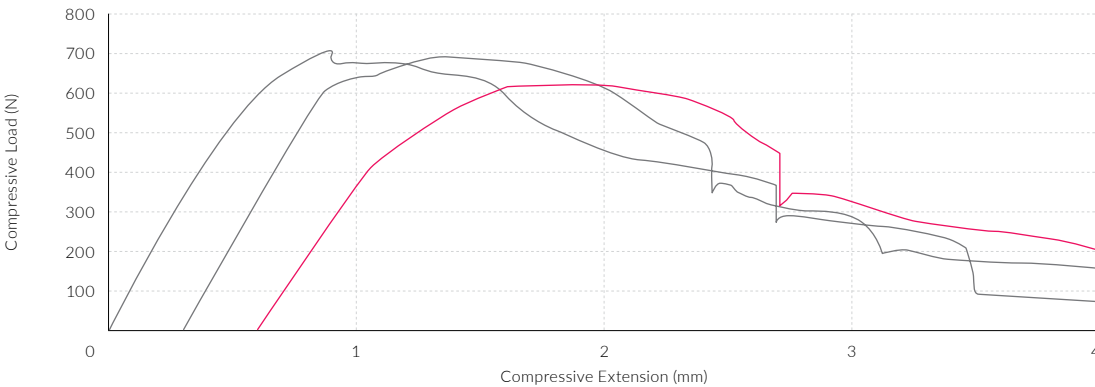

Ø 3.3

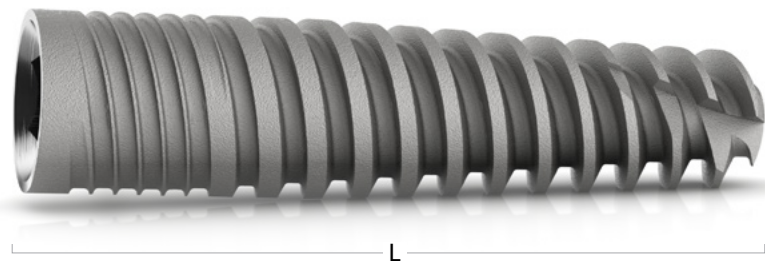

IA1-33010  
L10 mm



IA1-33011  
L11.5 mm




IA1-33013  
L13 mm



IA1-33016  
L16 mm

Ø 3.75

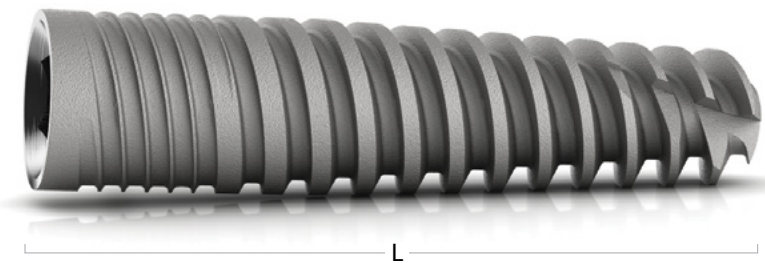

IA1-37508  
L8 mm



IA1-37510  
L10 mm



IA1-37511  
L11.5 mm



IA1-37513  
L13 mm



IA1-37516  
L16 mm

## Ø 3.3mm IMPLANT PROCEDURE

This procedure does not replace the professional judgment of the surgeon

\* Do not exceed torque more than 80 Ncm during implant insertion

| SOFT BONE |          |         | DENSE BONE |                   |
|-----------|----------|---------|------------|-------------------|
|           |          |         | * optional |                   |
|           |          |         |            |                   |
|           |          |         |            |                   |
| 1200-1500 | 900-1200 | 700-900 | 500-700    | Drill Speed (RPM) |
| Ø 1.90    | Ø 2.0    | Ø 2.5   | Ø 2.8      | Diameter          |


## Ø 3.75mm IMPLANT PROCEDURE

This procedure does not replace the professional judgment of the surgeon

| SOFT BONE |          |         | DENSE BONE |            |                   |
|-----------|----------|---------|------------|------------|-------------------|
|           |          |         | * optional |            |                   |
|           |          |         |            |            |                   |
|           |          |         |            |            |                   |
| 1200-1500 | 900-1200 | 500-700 | 400-700    | 200-400    | Drill Speed (RPM) |
| Ø 1.90    | Ø 2.0    | Ø 2.8   | C 3.2      | Ø 3.75/4.2 | Diameter          |



Ø4.2

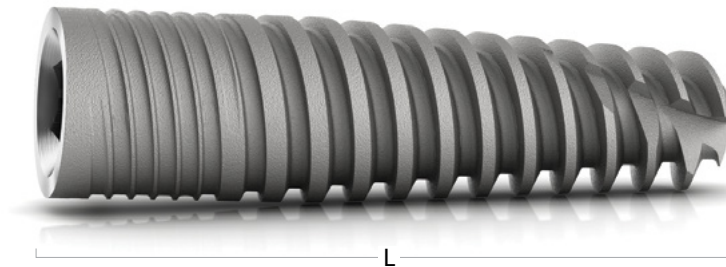


## Ø 4.2mm IMPLANT PROCEDURE

This procedure does not replace the professional judgment of the surgeon

| SOFT BONE                                                                           |                                                                                     |                                                                                     |                                                                                     | DENSE BONE                                                                                        |                                                                                                     |                                                                                       |
|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|
|  |  |  |  | * optional<br> | * optional<br> |  |
| 1200-1500                                                                           | 900-1200                                                                            | 500-700                                                                             | 400-700                                                                             | 400-600                                                                                           | 200-400                                                                                             | Drill Speed (RPM)                                                                     |
| Ø 1.90                                                                              | Ø 2.0                                                                               | Ø 2.8                                                                               | C 3.2                                                                               | C 3.8                                                                                             | Ø 3.5/4.2                                                                                           | Diameter                                                                              |

Ø5.0

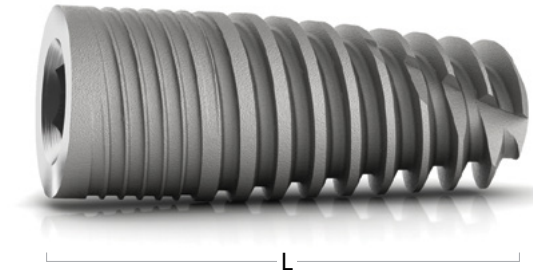


## Ø 5.0mm IMPLANT PROCEDURE

This procedure does not replace the professional judgment of the surgeon

| SOFT BONE                                                                             |                                                                                       |                                                                                       |                                                                                       |                                                                                       | DENSE BONE                                                                                          |                                                                                                     |                                                                                       |
|---------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|
|  |  |  |  |  | * optional<br> | * optional<br> |  |
| 1200-1500                                                                             | 900-1200                                                                              | 500-700                                                                               | 400-700                                                                               | 400-600                                                                               | 300-500                                                                                             | 200-400                                                                                             | Drill Speed (RPM)                                                                     |
| Ø 1.90                                                                                | Ø 2.0                                                                                 | Ø 2.8                                                                                 | C 3.2                                                                                 | C 3.8                                                                                 | C 4.5                                                                                               | Ø 5/6                                                                                               | Diameter                                                                              |

Ø 6.0

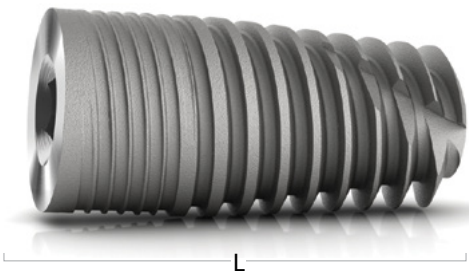

Ø 6.0mm IMPLANT PROCEDURE

This procedure does not replace the professional judgment of the surgeon

| SOFT BONE |          |         |         |         |         | DENSE BONE |            |                   |
|-----------|----------|---------|---------|---------|---------|------------|------------|-------------------|
|           |          |         |         |         |         | * optional | * optional |                   |
|           |          |         |         |         |         |            |            |                   |
| 1200-1500 | 900-1200 | 500-700 | 400-700 | 400-600 | 300-500 | 450-250    | 200-400    | Drill Speed (RPM) |
| Ø 1.90    | Ø 2.0    | Ø 2.8   | C 3.2   | C 3.8   | C 4.5   | C 5.5      | Ø 5/6      | Diameter          |





**MASSIF**  
INTERNAL HEX

# MASS|F Advantages



## CONNECTION

Internal hexagon 2.44mm.  
Provides antirotation.  
Provides positioning indication.  
Provides accurate and firm insertion using dedicated tooling.

## TOP

Micro rings - 4 micro rings at the implant crest module increase surface area for bone to implant contact. This improves primary stability and micro thread compressive strength in the crestal zone.

## GEOMETRY

A reverse buttress progressive thread allows for both bone compression and initial stability, and enables load transfer from crestal to the medullary bone.  
The double macro thread gives excellent primary stability in all bone types.

## SURFACE

The implant surface treatment provides optimal Ra values of 2 microns.


## APICAL

Self-tapping allows exceptional cutting capabilities, and is responsible for increased stability during insertion as the implant carves, fills, and compresses bone.


# MASS|F Implant Range

| Length (mm) | 6                                                                                                  | 8                                                                                                  | 10                                                                                                 | 11.5                                                                                               | 13                                                                                                 | 16                                                                                               |
|-------------|----------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------|
| Ø 3.75      |                                                                                                    | <br>IM1-37508   | <br>IM1-37510   | <br>IM1-37511   | <br>IM1-37513   | <br>IM1-37516 |
| Ø 4.2       | <br>IM1-42006   | <br>IM1-42008   | <br>IM1-42010   | <br>IM1-42011   | <br>IM1-42013   | <br>IM1-42016 |
| Ø 5.0       | <br>IM1-50006 | <br>IM1-50008 | <br>IM1-50010 | <br>IM1-50011 | <br>IM1-50013 |                                                                                                  |
| Ø 6.0       | <br>IM1-60006 | <br>IM1-60008 | <br>IM1-60010 | <br>IM1-60011 | <br>IM1-60013 |                                                                                                  |

Ø3.75

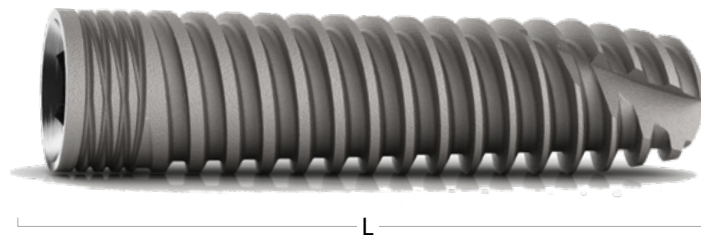

Ø4.2

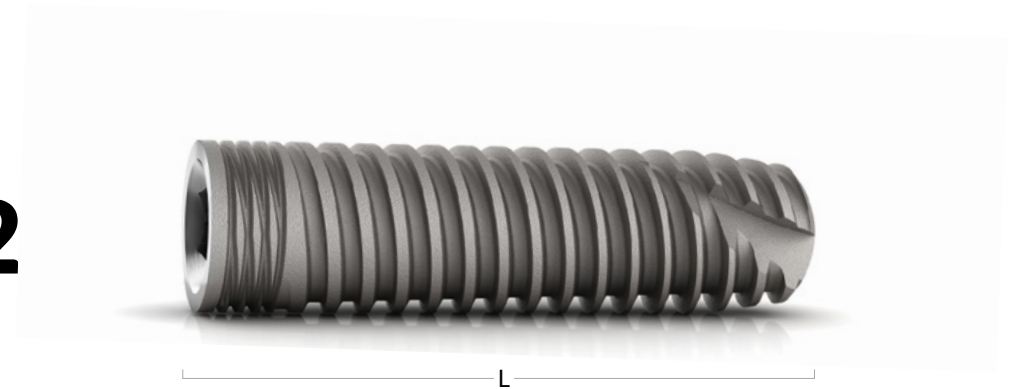

## Ø 3.75mm IMPLANT PROCEDURE

This procedure does not replace the professional judgment of the surgeon


| SOFT BONE |          |         | DENSE BONE     |                |                   |
|-----------|----------|---------|----------------|----------------|-------------------|
|           |          |         | * optional<br> | * optional<br> |                   |
| 1200-1500 | 900-1200 | 500-700 | 400-700        | 200-400        | Drill Speed (RPM) |
| Ø 1.90    | Ø 2.0    | Ø 2.8   | Ø 3.2          | Ø 3.75/4.2     | Diameter          |

## Ø 4.2mm IMPLANT PROCEDURE

This procedure does not replace the professional judgment of the surgeon

| SOFT BONE |          |         |         | DENSE BONE     |                |                   |
|-----------|----------|---------|---------|----------------|----------------|-------------------|
|           |          |         |         | * optional<br> | * optional<br> |                   |
| 1200-1500 | 900-1200 | 500-700 | 400-700 | 400-600        | 200-400        | Drill Speed (RPM) |
| Ø 1.90    | Ø 2.0    | Ø 2.8   | Ø 3.2   | Ø 3.8          | Ø 3.75/4.2     | Diameter          |

Ø5.0

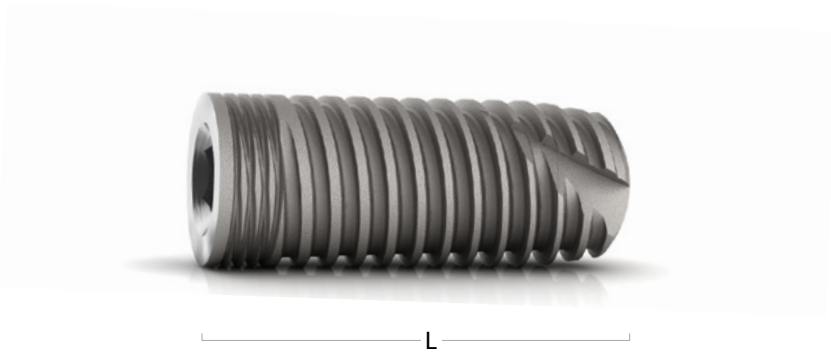


Ø 5.0mm IMPLANT PROCEDURE

This procedure does not replace the professional judgment of the surgeon

| SOFT BONE |          |         |         |         | DENSE BONE     |                |                   |
|-----------|----------|---------|---------|---------|----------------|----------------|-------------------|
|           |          |         |         |         | * optional<br> | * optional<br> |                   |
| 1200-1500 | 900-1200 | 500-700 | 400-700 | 400-600 | 300-500        | 200-400        | Drill Speed (RPM) |
| Ø 1.90    | Ø 2.0    | Ø 2.8   | Ø 3.2   | Ø 3.8   | Ø 4.5          | Ø 5/6          | Diameter          |

Ø6.0

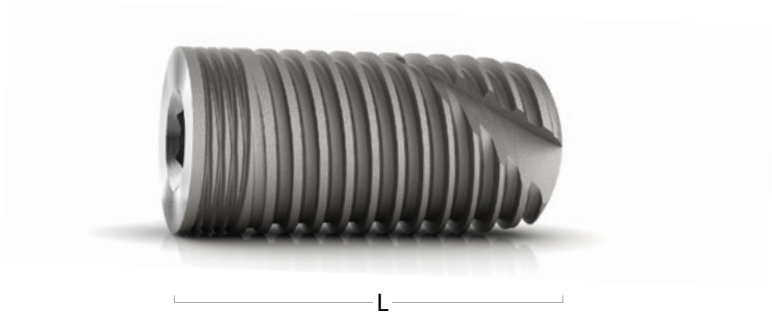

Ø 6.0mm IMPLANT PROCEDURE

This procedure does not replace the professional judgment of the surgeon

| SOFT BONE |          |         |         |         |         |         | DENSE BONE     |                |                   |
|-----------|----------|---------|---------|---------|---------|---------|----------------|----------------|-------------------|
|           |          |         |         |         |         |         | * optional<br> | * optional<br> |                   |
| 1200-1500 | 900-1200 | 500-700 | 400-700 | 400-600 | 300-500 | 200-400 | 200-450        | 200-400        | Drill Speed (RPM) |
| Ø 1.90    | Ø 2.0    | Ø 2.8   | Ø 3.2   | Ø 3.8   | Ø 4.5   | Ø 5     | Ø 5.5          | Ø 5/6          | Diameter          |





**CRESTONE**  
ONE PIECE

# CRESTONE Advantages



## CONNECTION

External 2.10mm square provides antirotation and accurate insertion using dedicated tooling.  
Provides positioning indication.

## TOP

The one piece Crestone implant design features an integrated abutment for single stage surgical procedures, and force transferring geometry.

## GEOMETRY

Innovative design with dual thread for easy insertion ensures maximum primary stability.

## SURFACE

The micro surface is achieved by blasting technology followed by acid etching.

## APICAL

Self-tapping allows for exceptional cutting capabilities, and is responsible for increased stability during insertion as the implant carves, fills, and compresses the bone.

## BIOCOMPATIBLE

Made of biocompatible Titanium Alloy Grade 23 (TI 6AL 4V ELI).  
Recommended use for the one piece Crestone implant is for limited edentulous ridge space. Available in several Lengths and Diameters.

# CRESTONE Implant Range

| Length (mm)     | 10                                                                                                | 11.5                                                                                              | 13                                                                                                | 16                                                                                                |
|-----------------|---------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------|
| Ø 3.0           | <br>IC1-30010  | <br>IC1-30011  | <br>IC1-30013  | <br>IC1-30016  |
|                 | <br>IC1-35010 | <br>IC1-35011 | <br>IC1-35013 | <br>IC1-35016 |
| Tools           | One Piece Analog 19mm                                                                             |                                                                                                   | Square Driver One Piece 28mm                                                                      |                                                                                                   |
| Stainless Steel | <br>PP1-0002 |                                                                                                   | <br>TK1-0011 |                                                                                                   |
|                 |                                                                                                   |                                                                                                   |                                                                                                   |                                                                                                   |

Ø3.0



IC1-30010  
L10 mm



IC1-30011  
L11.5 mm



IC1-30013  
L13 mm



IC1-30016  
L16 mm

Ø 3.0mm IMPLANT PROCEDURE

This procedure does not replace the professional judgment of the surgeon

| SOFT BONE |          |         | DENSE BONE     |                   |
|-----------|----------|---------|----------------|-------------------|
|           |          |         | <br>* optional |                   |
|           |          |         |                |                   |
|           |          |         |                |                   |
| 1200-1500 | 900-1200 | 700-900 | 500-700        | Drill Speed (RPM) |
| Ø 1.90    | Ø 2.0    | Ø 2.5   | Ø 2.8          | Diameter          |

Ø3.5



IC1-35010  
L10 mm



IC1-35011  
L11.5 mm



IC1-35013  
L13 mm



IC1-35016  
L16 mm

Ø 3.5mm IMPLANT PROCEDURE

This procedure does not replace the professional judgment of the surgeon

| SOFT BONE |          |         |         | DENSE BONE     |                   |
|-----------|----------|---------|---------|----------------|-------------------|
|           |          |         |         | <br>* optional |                   |
|           |          |         |         |                |                   |
|           |          |         |         |                |                   |
|           |          |         |         |                |                   |
| 1200-1500 | 900-1200 | 700-900 | 500-700 | 400-700        | Drill Speed (RPM) |
| Ø 1.90    | Ø 2.0    | Ø 2.5   | Ø 2.8   | Ø 3.2          | Diameter          |





restoration.


# Pre Restoration

## ONE IMPLANT ANALOG

L4 mm - Cat: PP1-0015

L6 mm - Cat: PP1-0001

L8 mm - Cat: PP1-00016

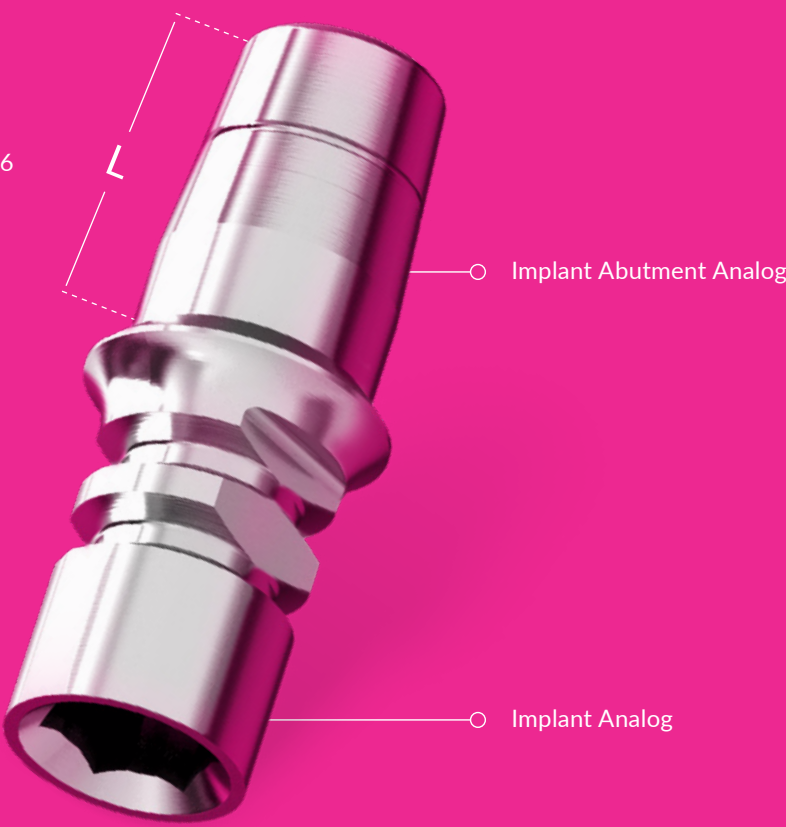


## TRANSFERS

| Height<br>Gingiva (mm) | 3                                                    | 5                                                  |
|------------------------|------------------------------------------------------|----------------------------------------------------|
| Open Tray              | <br>PP1-0003<br>Titanium & Stainless Steel<br>L10 mm | <br>PP1-0010<br>Stainless Steel<br>L16 mm          |
| Close Tray             | <br>PP1-0005<br>Stainless Steel<br>L12 mm            | <br>PP1-0012<br>Titanium&Stainless Steel<br>L14 mm |
| Press Fit              | <br>PP1-0009<br>Stainless Steel<br>L12 mm            | <br>PP1-0014<br>Stainless Steel<br>L14 mm          |

# Pre Restoration

## HEALING CAPS

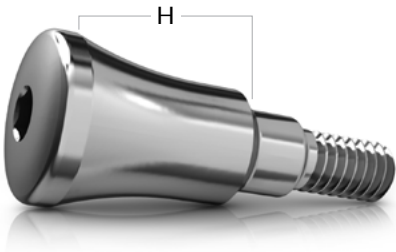

Ø 4  
Straight



Ø 5  
Straight



Ø 5.8  
Straight



Ø 5  
Anatomic




Ø 5.8  
Wide Anatomic



# Temporary Restoration

## PEEK ESTHETIC ABUTMENTS

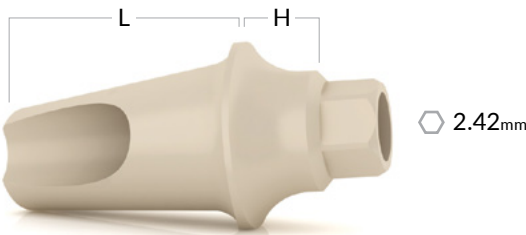

### Temporary Abutment

0°  
Open Tray



15°



25°




## SCREWS

All abutments are supplied with the relevant screw.  
The titanium Abutment screw should be tightened to 30-35 Ncm.



# Final Restoration

## TITANIUM ANATOMIC ABUTMENTS

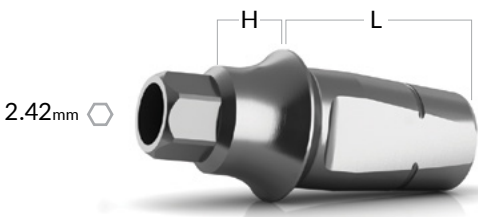

L4



PF1-0005  
H1 mm



PF1-0008  
H2 mm



PF1-0011  
H3 mm



PF1-0014  
H4 mm



PF1-0087  
H5 mm

L6



PF1-0006  
H1 mm



PF1-0009  
H2 mm



PF1-0012  
H3 mm



PF1-0015  
H4 mm



PF1-0088  
H5 mm

L8



PF1-0007  
H1 mm



PF1-0010  
H2 mm



PF1-0013  
H3 mm



PF1-0016  
H4 mm



PF1-0089  
H5 mm

L6

Wide Platform



PF1-0065  
H2 mm



PF1-0064  
H3 mm



PF1-0063  
H4 mm



PF1-0090  
H5 mm

### SCREWS

All abutments are supplied with the relevant screw.  
The titanium Abutment screw should be tightened  
to 30-35 Ncm.




PF1-0002



PF1-0001

## TITANIUM ESTHETIC ABUTMENTS

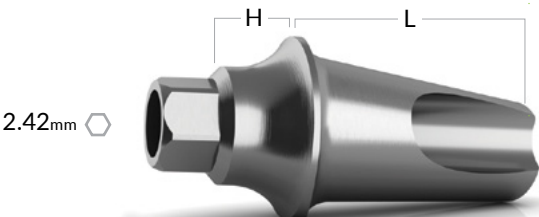

0°

Straight



PF1-0082  
Narrow Solid  
H0/L8 mm



PF1-0069  
Narrow  
H0/L8 mm



PF1-0026  
Esthetic  
H1/L9 mm



PF1-0027  
Esthetic  
H2/L9 mm



PF1-0028  
Esthetic  
H3/L9 mm

15°



PF1-0073  
Narrow  
H0/L9 mm



PF1-0024  
Angulated  
L9 mm



PF1-0029  
Esthetic  
H1/L9 mm



PF1-0030  
Esthetic  
H2/L9 mm



PF1-0031  
Esthetic  
H3/L9 mm



PF1-0066  
Angulated  
Wide Platform  
H2/L9 mm

25°



PF1-0077  
Narrow  
H0/L9 mm



PF1-0025  
Angulated  
L9 mm



PF1-0032  
Esthetic  
H1/L8 mm



PF1-0033  
Esthetic  
H2/L8 mm



PF1-0034  
Esthetic  
H3/L8 mm



PF1-0067  
Angulated  
Wide Platform  
H2/L9 mm

### Universal



PF1-0081  
H0/L11 mm



PF1-0023  
H2/L9 mm



PF1-0068  
Wide Platform  
H2/L9 mm

### SCREWS

All abutments are supplied with the relevant screw.  
The titanium Abutment screw should be tightened  
to 30-35 Ncm.



PF1-0002



PF1-0001

# Final Restoration

## PLASTIC CYLINDER ABUTMENTS

Hexagon Round



L11

Direct



PF1-0035  
Hexagon Acetal



PF1-0036  
Round Acetal

L15

Titanium



PF1-0058  
Hexagon Acetal, Titanium



PF1-0057  
Round Acetal, Titanium

L15

Direct Gold



PF1-0059  
Hexagon Acetal, Gold



PF1-0060  
Round Acetal, Gold

L15

Direct CoCr



PF1-0092  
Hexagon Acetal, CoCr



PF1-0091  
Round Acetal, CoCr

### SCREWS

All abutments are supplied with the relevant screw.  
The titanium Abutment screw should be tightened to 30-35 Ncm.




PF1-0002



PF1-0001

## BURN-OUT ESTHETIC ABUTMENTS

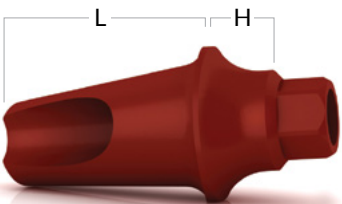

2.42mm

0°

L9 mm



PF1-0037  
H1 mm / Acetal



PF1-0039  
H2 mm / Acetal



PF1-0040  
H3 mm / Acetal

15°

L9 mm



PF1-0041  
H1 mm / Acetal



PF1-0042  
H2 mm / Acetal



PF1-0043  
H3 mm / Acetal

25°

L8 mm



PF1-0044  
H1 mm / Acetal



PF1-0045  
H2 mm / Acetal



PF1-0046  
H3 mm / Acetal

### SCREWS

All abutments are supplied with the relevant screw.  
The titanium Abutment screw should be tightened to 30-35 Ncm.




PF1-0002



PF1-0001

# Final Restoration

## ANATOMIC RESTORATION KIT

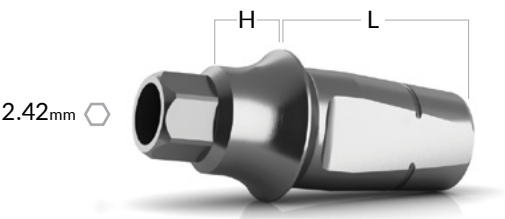

|    |                    |                    |                    |                    |                    |
|----|--------------------|--------------------|--------------------|--------------------|--------------------|
| L4 | PF1-ARK14<br>H1 mm | PF1-ARK24<br>H2 mm | PF1-ARK34<br>H3 mm | PF1-ARK44<br>H4 mm | PF1-ARK54<br>H5 mm |
| L6 | PF1-ARK16<br>H1 mm | PF1-ARK26<br>H2 mm | PF1-ARK36<br>H3 mm | PF1-ARK46<br>H4 mm | PF1-ARK56<br>H5 mm |
| L8 | PF1-ARK18<br>H1 mm | PF1-ARK28<br>H2 mm | PF1-ARK38<br>H3 mm | PF1-ARK48<br>H4 mm | PF1-ARK58<br>H5 mm |



# Overdenture Abutments


## BALL ATTACHMENT ANGULATED (BAA) - PATENT PENDING

The BAA system provides the ability to restore even the most compromised of edentulous cases. Made in one piece the BAA is an esthetic design with reduced size that improves the overall esthetics of the restoration.



## MULTI UNIT ANGULATED (MUA) - PATENT PENDING

Made in one piece the MUA has an esthetic and compact design. Reduced size improves the esthetic restoration.

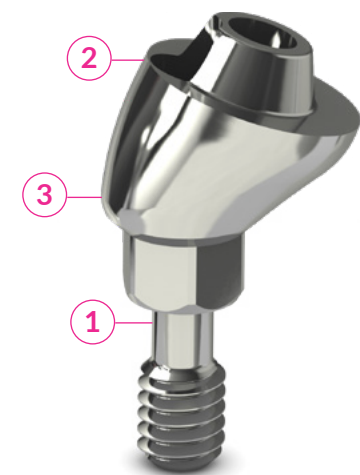


- 1 One-piece Design
- 2 Reduced Size Design
- 3 Intelligent Design which Allows for Restorative Flexibility

# MAXI Angulation MU


## SOLUTIONS FOR EXTREME SITUATIONS

MUA and BAA are intended for use with implants of diameter 3.3mm to 6mm

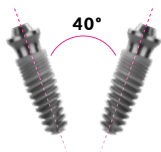


### Insertion Path

Straight Multi-Unit abutments can be used with implants with divergence of up to 40°

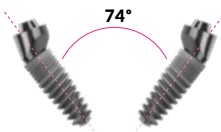


### Insertion Path: MUA = 17

17° Multi-Unit abutments can be used with implants with divergence of up to 74°

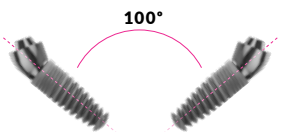

### MUA = 30

30° Multi-Unit abutments can be used with divergence of up to 100°

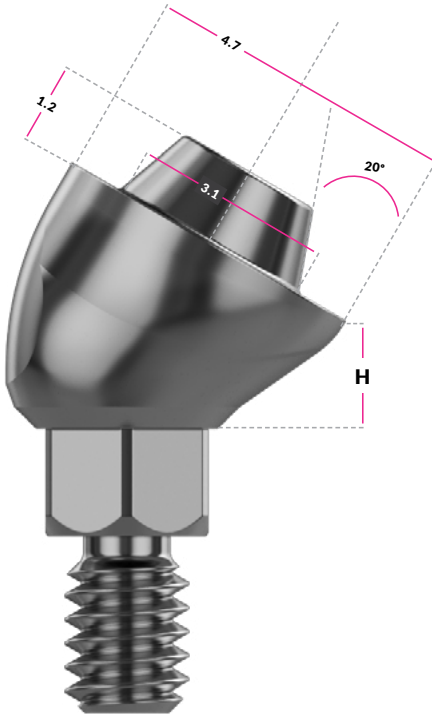


Key for M.U & balls  
TK1 - 0033  
Stainless Steel

Stem  
TK1-0039

# Multi-Unit

## MULTI-UNIT ABUTMENTS

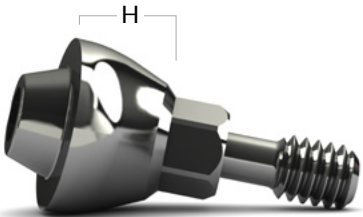

Titanium

**PO1-0016**  
H1 mm

**PO1-0017**  
H2 mm

**PO1-0018**  
H3 mm

**PO1-0019**  
H4 mm

**PO1-0020**  
H5 mm

**PO1-0038**  
17° / H2 mm

**PO1-0039**  
17° / H3 mm

**PO1-0047**  
17° / H4 mm

**PO1-0040**  
30° / H2 mm

**PO1-0041**  
30° / H3 mm

**PO1-0049**  
30° / H4 mm

PATENT PENDING

## ACCESSORIES

**PO1-0027**  
Plastic Cylinder  
L10 mm / Acetal

**PO1-0028**  
Temporary Ti  
L10 mm / Titanium

**PO1-0024**  
Healing Cap  
L3.5 mm / Titanium

**PO1-0021**  
Abutment Analog  
L12 mm / Stainless Steel

**TK1-0039**  
Angulated Multi Unit Stem

**TK1-0033**  
Key for M.U&Balls  
Stainless Steel

**PO1-0031**  
Impression Coping Open Tray  
L10 mm / Stainless Steel

**PO1-0033**  
Impression Coping Close tray  
L6 mm / Stainless Steel


## SCREWS

**PO1-0046**  
All MUA are supplied with an abutment screw. Tighten the screw to 30Ncm.

**PO1-0025**  
All Multi-Unit Accessories are supplied with screw. MUA prosthetic screw tightens to 30Ncm.

# Ball Attachments

## BALL ATTACHMENTS ABUTMENTS

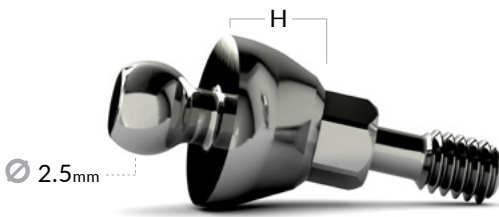

Titanium

**PO1-0001**  
H1 mm

**PO1-0002**  
H2 mm

**PO1-0003**  
H3 mm

**PO1-0004**  
H4 mm

**PO1-0005**  
H5 mm

**PO1-0042**  
17° / H2 mm

**PO1-0043**  
17° / H3 mm

**PO1-0051**  
17° / H4 mm

**PO1-0044**  
30° / H2 mm

**PO1-0045**  
30° / H3 mm

**PO1-0053**  
30° / H4 mm

PATENT PENDING

## ACCESSORIES

**TK1-0033**  
Key for M.U&balls  
Stainless Steel

**PO1-0006**  
Ball Attachment Analog  
L12 mm / Stainless Steel

**PO1-0009**  
Metal Housing for Plastic Ball Cap

**PO1-0012**  
Soft Plastic Cap Ball  
900gr

**PO1-0013**  
Standard Plastic Cap Ball  
1300gr

**PO1-0014**  
Plastic Disc for Ball Attachment

## SCREW

All our Ball Attachment Angulated (BAA) are supplied with an abutment. Tighten the screw to 30Ncm.

**PO1-0046**

# Equator

## EQUATOR ATTACHMENT




### Equator Attachments

- PO1-0056 H1 mm
- PO1-0057 H2 mm
- PO1-0058 H3 mm
- PO1-0059 H4 mm
- PO1-0060 H5 mm

## EQUATOR ABUTMENT



## EQUATOR ATTACHMENT ACCESSORIES

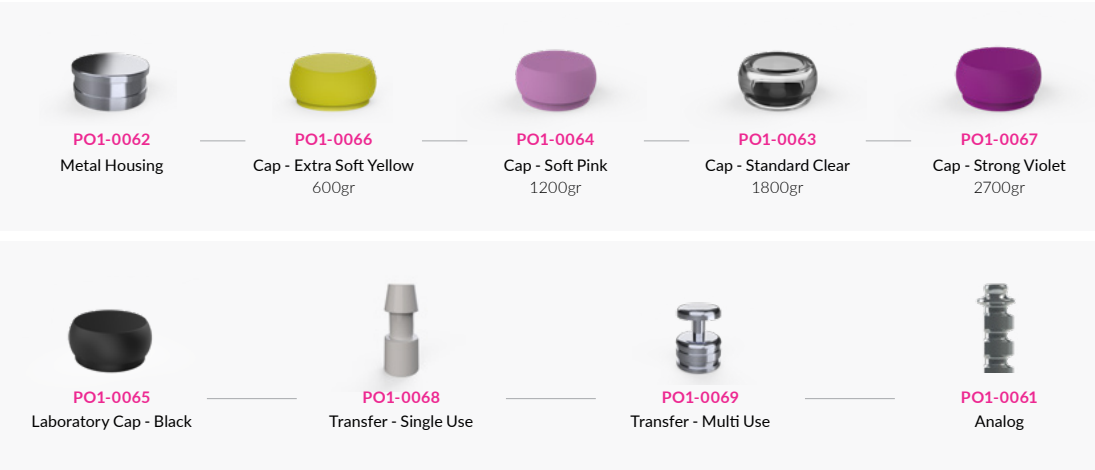

## TOOLS



TK1-0035  
Equator Cap Insertion Tool



TK1-0035  
Plastic Disc for Ball Attachment



instruments.

# Surgical Instruments



## DRILLS

TAG drills are compatible with all TAG implants. The externally irrigated drills are supplied as either standard or short, and drill diameter can be accurately identified through the simple color coding system.

## SURGICAL TOOLS

TAG produces superior quality and attractively designed tools. Surgical tools are not sterilized prior to shipping. The tools must be cleaned, disinfected and sterilized before use. After sterilization, check the packaging of the sterilized instruments for damage.

Check the sterilization indicators.

Store tools in a dry, dust-free, vapor-free area at a moderate temperature. See full instructions for use accompanying each kit.



## SURGICAL KITS

TAG offers various types of surgical kits designed to provide complete solutions for implantation procedures.

Kits contain tooling and drills for implantation of 3.3-6.0mm implants.





# Surgical Kits

**PREMIUM  
SURGICAL KIT**  
**KT1-0001**



**Drills**



**TD1-0002**  
Marking Drill  
Ø1.90 mm



**TD1-0003**  
Pilot Drill  
External Irrigation  
Ø2 mm



**TD1-0004**  
Pilot Drill  
External Irrigation  
Ø2.5 mm



**TD1-0005**  
Pilot Drill  
External Irrigation  
Ø2.80 mm



**TD1-0006**  
Pilot Drill  
External Irrigation  
Ø3.20 mm



**TD1-0007**  
Pilot Drill  
External Irrigation  
Ø3.80 mm



**TD1-0008**  
Drill  
External Irrigation  
Ø4.50 mm



**TD1-0009**  
Drill  
External Irrigation  
Ø5.00 mm



**TD1-0010**  
Drill  
External Irrigation  
Ø5.50 mm



**TD1-0021**  
Conical Drill  
for implant  
Ø3.75 mm



**TD1-0022**  
Conical Drill  
for implant  
Ø4.2 mm



**TD1-0023**  
Conical Drill  
for implant  
Ø5 mm



**TD1-0024**  
Conical Drill  
for implant  
Ø6 mm



**TD1-0030**  
Drill Extender

**Screw Drivers**  
Stainless Steel



**TK1-0008**  
Motor key 1.28 mm  
Long / 28 mm



**TK1-0007**  
Motor key 1.28 mm  
Short / 22 mm



**TK1-0001**  
Key 1.28 mm  
Short / 15 mm



**TK1-0002**  
Key 1.28 mm  
Long / 23 mm



**TK1-0033**  
Key for M.U & Balls

**Implant Drivers**  
Stainless Steel



**TK1-0003**  
Key 2.42 mm  
Short / 15 mm



**TK1-0004**  
Key 2.42 mm  
Long / 23 mm



**TK1-0011**  
Square Driver  
One Piece / 28 mm



**TK1-0018**  
Hand Tool 2.42 mm  
Long / 18 mm



**TK1-0019**  
Retriever 2.42 mm  
15 mm



**TK1-0009**  
Motor key 2.42 mm  
Short / 20 mm



**TK1-0010**  
Motor key 2.42 mm  
Long / 28 mm

**TOOLS**



**TD1-0019**  
Countersink  
Ø3.75-4.2 mm




**TD1-0020**  
Countersink  
Ø5-6 mm




**TK1-0012**  
Parallel pin  
L11 mm  
Stainless Steel x3



**TK1-0013**  
Implant indicator  
L14 mm  
Stainless Steel x2

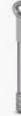

**TK1-0015**  
Ratchet  
L85 mm  
Stainless Steel

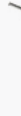

**TK1-0014**  
Depth Probe



# Surgical Kits

## COMPACT AXIS SURGICAL KIT

KT1-0006



### Drills



**TD1-0002**  
Marking Drill  
Ø1.90 mm



**TD1-0003**  
Pilot Drill  
External Irrigation  
Ø2 mm



**TD1-0004**  
Pilot Drill  
External Irrigation  
Ø2.5 mm



**TD1-0005**  
Drill  
External Irrigation  
Ø2.80 mm



**TD1-0021**  
Conical Drill for implant  
Ø3.75 mm



**TD1-0022**  
Conical Drill for implant  
Ø4.2 mm



**TD1-0023**  
Conical Drill for implant  
Ø5 mm



**TD1-0024**  
Conical Drill for implant  
Ø 6mm

### Screw Drivers Stainless Steel



**TK1-0002**  
Key 1.28mm  
Long / 23 mm



**TK1-0004**  
Key 2.42mm  
Long / 23 mm



**TK1-0010**  
Motor key 2.42mm  
Long / 28 mm



**TK1-0019**  
2.42 Retriever  
15 mm / Stainless Steel, Titanium

**TOOLS**



**TD1-0019**  
Countersink  
Ø3.75-4.2 mm



**TD1-0020**  
Countersink  
Ø5-6 mm



**TK1-0012**  
Parallel pin  
Ø 1.90mm / L11 mm  
Stainless Steel



**TK1-0013**  
Implant indicator  
Ø 1.90mm / L14 mm  
Stainless Steel



**TK1-0015**  
Ratchet  
Ø 1.90mm / L84 mm  
Stainless Steel

## COMPACT MASSIF SURGICAL KIT

KT1-0002



### Drills



**TD1-0002**  
Marking Drill  
Ø1.90 mm



**TD1-0003**  
Pilot Drill  
External Irrigation  
Ø2 mm



**TD1-0005**  
Drill  
External Irrigation  
Ø2.80 mm



**TD1-0006**  
Drill  
External Irrigation  
Ø3.20 mm



**TD1-0007**  
Pilot Drill  
External Irrigation  
Ø3.80 mm



**TD1-0008**  
Drill  
External Irrigation  
Ø4.50 mm



**TD1-0009**  
Drill  
External Irrigation  
Ø5.00 mm



**TD1-0010**  
Drill  
External Irrigation  
Ø5.50 mm

### Screw Drivers Stainless Steel



**TK1-0002**  
Key 1.28mm  
Long / 23 mm



**TK1-0004**  
Key 2.42mm  
Long / 23 mm



**TK1-0010**  
Motor key 2.42mm  
Long / 28 mm



**TK1-0019**  
2.42 Retriever  
15 mm / Stainless Steel, Titanium

**TOOLS**



**TD1-0019**  
Countersink  
Ø3.75-4.2 mm



**TD1-0020**  
Countersink  
Ø5-6 mm



**TK1-0012**  
Parallel pin  
Ø 1.90mm / L11 mm  
Stainless Steel



**TK1-0013**  
Implant indicator  
Ø 1.90mm / L14 mm  
Stainless Steel



**TK1-0015**  
Ratchet  
Ø 1.90mm / L84 mm  
Stainless Steel

# Surgical Kits

## PIXEL CRESTONE SURGICAL KIT KT1-0007



### Drills

- 

**TD1-0002**  
Marking Drill  
Ø1.90 mm
- 

**TD1-0003**  
Pilot Drill  
External Irrigation  
Ø2 mm
- 

**TD1-0004**  
Pilot Drill  
External Irrigation  
Ø2.5 mm
- 

**TD1-0005**  
Drill  
External Irrigation  
Ø2.80 mm
- 

**TD1-0006**  
Drill  
External Irrigation  
Ø3.20 mm

### TOOLS Stainless Steel

- 

**TK1-0011**  
Square Driver One Piece  
28 mm

- 

**TK1-0012**  
Parallel pin  
L11 mm

- 

**TK1-0015**  
Ratchet  
L84 mm

# Prosthetic Kit

## PROSTHETIC KIT KT1-0008



- 

**TK1-0001**  
Key 1.28mm  
Short / L15 mm  
Stainless Steel
- 

**TK1-0002**  
Key 1.28mm  
Long / L23 mm  
Stainless Steel
- 

**TK1-0033**  
Key for M.U & Balls  
Stainless Steel
- 

**TK1-0007**  
Motor key 1.28  
Short / L22 mm  
Stainless Steel
- 


**TK1-0008**  
Motor key 1.28  
Long / L28 mm  
Stainless Steel
- 

**TK1-0013**  
Implant indicator  
L14 mm  
Stainless Steel
- 

**TK1-0020**  
Abutment  
Extractor Key
- 

**TK1-0016**  
Torque Ratchet  
L85 mm / 15-45 Ncm  
Stainless Steel
- 

**TK1-0006**  
Screw Driver 1.28mm  
Long / L25 mm  
Stainless Steel
- 

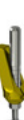**TK1-0030**  
Mock Abutment  
Straight
- 

**TK1-0031**  
Mock Abutment  
15°
- 

**TK1-0032**  
Mock Abutment  
25°

## SCREW REMOVAL KIT KT1-0009

- 

**TK1-0036**  
Screw Removal  
Burr Guide
- 

**TK1-0037**  
1-72 UNF Tap
- 

**TK1-0038**  
Screw Removal  
Burr



# Holding Kits

## UNIVERSAL HOLDING KIT

KT1-0004



TK1-0034  
Handle Set  
100 mm / Stainless Steel



TK1-0027  
Handle Key 1.28mm  
43 mm / Stainless Steel



TK1-0024  
Extension for 1/4" Holder  
50 mm / Stainless Steel



TK1-0028  
Handle Key 2.42mm  
43 mm / Stainless Steel



TK2-0011  
CC Handle Key  
2.42 mm

## TECHNICIAN HOLDING KIT

KT1-0005



TK1-0034  
Handle Set  
100 mm / Stainless Steel



TK1-0023  
1.28 Kit Screwdriver  
140 mm / Stainless Steel



TK1-0025  
1/4" Abutment Fixture  
Long / 15 mm  
Stainless Steel



TK2-0010  
CC 1/4" Abutment Fixture  
Long



# Drills & Stoppers



Ø 3.8



TD1-0015  
Short Drill Ø 3.80mm



TD1-0007  
Drill Ø 3.80mm



\*  
Drill Stopper

Length \*

|                   |                   |                    |                      |                    |                    |
|-------------------|-------------------|--------------------|----------------------|--------------------|--------------------|
| TD2-0038<br>L 6mm | TD2-0046<br>L 8mm | TD2-0054<br>L 10mm | TD2-0062<br>L 11.5mm | TD2-0070<br>L 13mm | TD2-0078<br>L 16mm |
|-------------------|-------------------|--------------------|----------------------|--------------------|--------------------|

Ø 4.5



TD1-0016  
Short Drill Ø 4.50mm



TD1-0008  
Drill Ø 4.50mm



\*  
Drill Stopper

Length \*

|                   |                   |                    |                      |                    |                    |
|-------------------|-------------------|--------------------|----------------------|--------------------|--------------------|
| TD2-0039<br>L 6mm | TD2-0047<br>L 8mm | TD2-0055<br>L 10mm | TD2-0063<br>L 11.5mm | TD2-0071<br>L 13mm | TD2-0079<br>L 16mm |
|-------------------|-------------------|--------------------|----------------------|--------------------|--------------------|

Ø 5



TD1-0017  
Short Drill Ø 5mm



TD1-0009  
Drill Ø 5mm



\*  
Drill Stopper

Length \*

|                   |                   |                    |                      |                    |                    |
|-------------------|-------------------|--------------------|----------------------|--------------------|--------------------|
| TD2-0040<br>L 6mm | TD2-0048<br>L 8mm | TD2-0056<br>L 10mm | TD2-0064<br>L 11.5mm | TD2-0072<br>L 13mm | TD2-0080<br>L 16mm |
|-------------------|-------------------|--------------------|----------------------|--------------------|--------------------|

Ø 5.5



TD1-0018  
Short Drill Ø 5.50mm



TD1-0010  
Drill Ø 5.50mm



\*  
Drill Stopper

Length \*

|                   |                   |                    |                      |                    |                    |
|-------------------|-------------------|--------------------|----------------------|--------------------|--------------------|
| TD2-0041<br>L 6mm | TD2-0049<br>L 8mm | TD2-0057<br>L 10mm | TD2-0065<br>L 11.5mm | TD2-0073<br>L 13mm | TD2-0081<br>L 16mm |
|-------------------|-------------------|--------------------|----------------------|--------------------|--------------------|

\*Drills stoppers available only for Massif standard length. All drills are External irrigation & made of stainless steel

## MARKING DRILLS



TD1-0001  
Spade Marking Drill  
Ø 1.50mm



TD1-0002  
Marking Drill  
Ø 1.90mm

## COUNTERSINK DRILLS



TD1-0019  
Countersink  
Ø 3.75-4.20mm



TD1-0020  
Countersink  
Ø 5-6mm



## CONICAL DRILLS



TD1-0021  
Conical Drill for implant  
Ø 3.75mm



TD1-0022  
Conical Drill for implant  
Ø 4.20mm



TD1-0023  
Conical Drill for implant  
Ø 5mm



TD1-0024  
Conical Drill for implant  
Ø 6mm

# Surgical Tools

## GROOVED

Providing firm, accurate and easy hand operation screwing and un-screwing

## OVERSIZED

Providing easy extraction from ratchet wrench. Laser Marked for easy identification

## PARALLEL 3.1mm HEXAGON

Provides external visualization of internal hex position



## 6.35mm (1/4") HEXAGON

Fits to T.A.G.'s dedicated torque and regular ratchet wrench

## 2.42mm HEXAGON

Allows secure removal of implant from sterile vial

## IMPLANT DRIVERS

Stainless Steel

|                                                                                                                                              |                                                                                                                                             |                                                                                                                                                      |                                                                                                                                           |
|----------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------|
| <br><b>TK1-0003</b><br>Key 2.42mm<br>Short / 15mm         | <br><b>TK1-0004</b><br>Key 2.42mm<br>Long / 23mm         | <br><b>TK1-0009</b><br>Motor Key 2.42mm<br>Short / 20mm           | <br><b>TK1-0010</b><br>Motor key 2.42mm<br>Long / 28mm |
| <br><b>TK1-0017</b><br>Hand Tool 2.42mm<br>Short / L 10mm | <br><b>TK1-0018</b><br>Hand Tool 2.42mm<br>Long / L 18mm | <br><b>TK1-0019</b><br>2.42mm Retriever<br>L 15mm / S.S, Titanium | <br><b>TK1-0011</b><br>Square Driver One Piece<br>28mm |

## SCREW DRIVERS

Stainless Steel

|                                                                                                                                           |                                                                                                                                                 |                                                                                                                                                |                                                                                                                               |
|-------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------|
| <br><b>TK1-0001</b><br>Key 1.28mm<br>Short / 15mm    | <br><b>TK1-0002</b><br>Key 1.28mm<br>Long / 23mm           | <br><b>TK1-0007</b><br>Motor key 1.28mm<br>Short / 22mm   | <br><b>TK1-0033</b><br>Key for M.U&balls |
| <br><b>TK1-0008</b><br>Motor key 1.28<br>Long / 28mm | <br><b>TK1-0005</b><br>Screw Driver 1.28mm<br>Short / 15mm | <br><b>TK1-0006</b><br>Screw Driver 1.28mm<br>Long / 25mm |                                                                                                                               |



# Surgical Tools

## INSTRUMENTS

Stainless Steel



TD1-0030  
Drill Extender  
30mm



TK1-0012  
Parallel Pin  
Ø 2-3.20mm / L 11mm



TK1-0013  
Implant Indicator  
14mm



TK1-0015  
Ratchet  
85mm



TK1-0016  
Torque Ratchet  
85mm / 15-45 Ncm



TK1-0014  
Depth Probe



TK1-0020  
Abutment Extractor Key  
22mm

## TREPHINE BURRS



TD1-0031  
Trephine Burrs  
Ø 3x4mm



TD1-0032  
Trephine Burrs  
Ø 4x5mm



TD1-0033  
Trephine Burrs  
Ø 5x6mm





# SYBONE

SYNTHETIC BONE GRAFT SUBSTITUTE

# Introducing SYBONE



## FROM THE ARTIFICIAL TO THE LIVING BONE

SYBONE is an osteoconductive synthetic graft material designed to fill the space of the missing bone and encourage the growth of new bone tissue.

Vial = 1gr (0.9cc) (5 vials in the box)

## ADVANTAGES

- 100% synthetic & 100% resorbable.
- Sybone is replaced by new vital bone within 6 months.
- Allows the perfect monitorization of osteointegration.
- High Cohesiveness and volume conservation: Sybone is very hydrophilic, that confers an excellent cohesivity of particules, which in turn allows the conservation of the volume of the initial cavity.
- Sybone induces a remarkable vascularization.

## ALLOPLAST = SYNTHETIC ORIGIN

**Composition**  
99.9% Tricalcium Phosphate (3-TCP)

**Proprieties**  
Biocompatibility  
Osteoconductive  
Optimal Resorption Rate  
Excellent Mechanical Resistance

**Indications**  
SYBONE is intended to be used as a bone void filler or augmentation material for bone defects. Sinus floor elevation Alveolar filling or augmentation Alveolar regeneration Reconstruction of tumor and cyst defects Filling of extraction cavities.

# Bone

## INSTRUCTIONS FOR USE



Impregnate the implant with patient's blood or autologous bone marrow.

Sybone should have contact with cancellous tissue.

The bone surface must be freshened and slightly bleeding. The filling must be complete with slight impaction.

The wound closure must be complete and airtight.

**ABM-0002**  
Granules  
0.1-0.5mm / 1gr.



Indicated for small periodontal defects

**ABM-0003**  
Granules  
0.5-1mm / 1gr.



Indicated for the filling of cysts and the filling of alveolar cavity

**ABM-0004**  
Granules  
1-2mm / 1gr.



Indicated for sinus floor elevation and the filling of large cysts



# SYCURE

RESORBABLE SYNTHETIC MEMBRANE

# Resorbable & Biocompatible Synthetic Membrane

## ADVANTAGE

Medical-grade Polylactic-Glycolic Acid (PLGA) provides an excellent biocompatibility.

SyCure membrane is fully controlled resorption and a 100% biodegradable polymer.

The barrier function of SyCure membrane remains intact for the first 4 weeks.

Optimal bone and tissue regeneration are both guaranteed thanks to the slow fully controlled resorption over 6 months, avoiding any need for second stage surgery for membrane removal.


## BILAYER STRUCTURE FOR OPTIMAL BARRIER EFFECT

### Smooth side of dense layer

Prevent (gingival) epithelial tissue ingrowth while promoting cell infiltration.

### Matt side with non-woven microfibers

Guided bone healing.

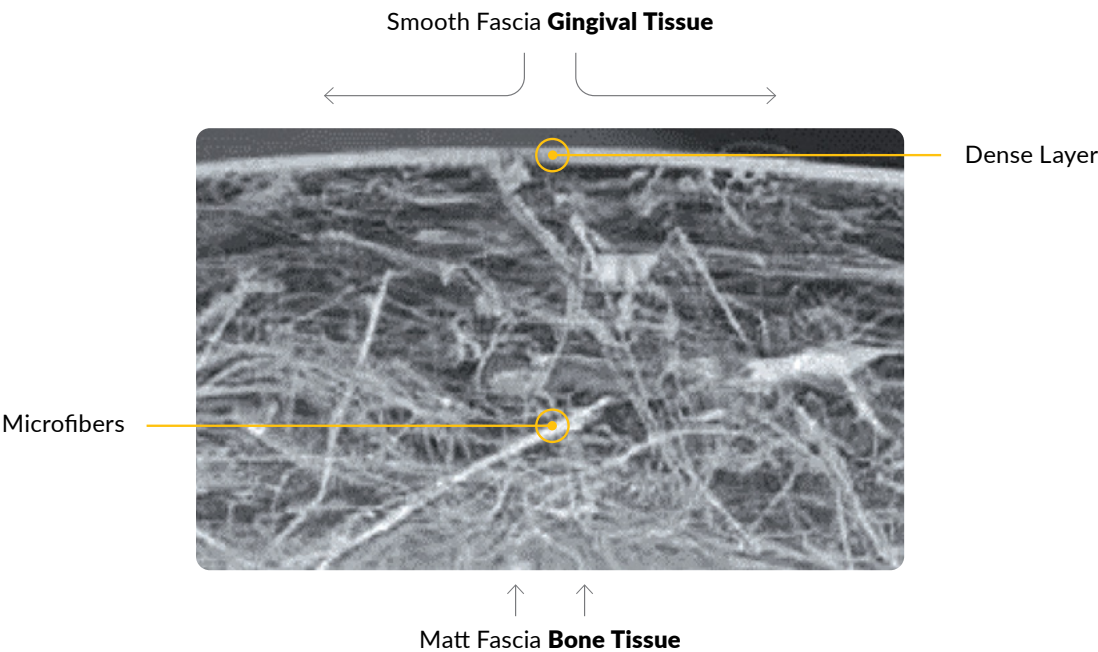

## EASY TO USE

- Does not stick to soft tissue or instruments.
- No need for prior wetting, pinning or suturing.
- Quickly soaks up biological fluids on the micro-fiber side.
- Once wet SyCure respects the form and shape that you create.
- Tear resistant for tacking and suturing.
- Easy to cut.
- SyCure is free from animal derivatives.
- Our biocompatible synthetic membrane avoids the risk of transmission of animal pathogens.
- Prevent the proliferation of epithelial cells and encourage the migration of bone cells within the blood clot.
- Keep the bone graft and blood clot in place.
- To help prevent bone loss by as much as 25% (Widmark et al., 1997).
- SyCure membranes are suitable for patients who avoid animal by-products for cultural reasons or lifestyle choices.



The microfibres rapidly absorb the biological fluids. The membrane respects the configuration and the shape required and adheres perfectly to the surgical site.



# Characteristics

## COMPOSITION

Synthetic PLGA (Animal free)

## RESORPTION TIME

4 to 6 months

## SUTURES, PINS

Not necessary

## LAYERS 2 BARRIER EFFECT

4 weeks

## MECHANICS

Maintains the bone graft in place

## EXPIRY DATE

36 months

## STORAGE

15 to 25 oC

## PACKAGING


Outer box plus double alum. sachets, easier to keep sterile

## IN CASE OF EXPOSURE

Reepithelialisation over 4 weeks




## AVAILABLE IN 4 DIFFERENT SIZES

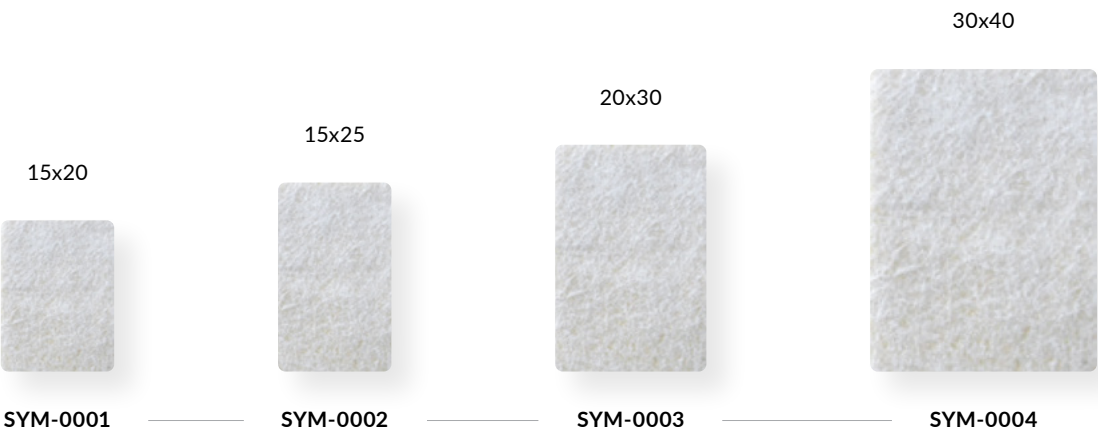

Our advanced tissue technology results in improved bioresorbable polymer-based membrane for tissue regeneration.

Contact us to find out more about how TAG Dental SYCURE’s ongoing research and development in regenerative medicine can help you better serve patients.

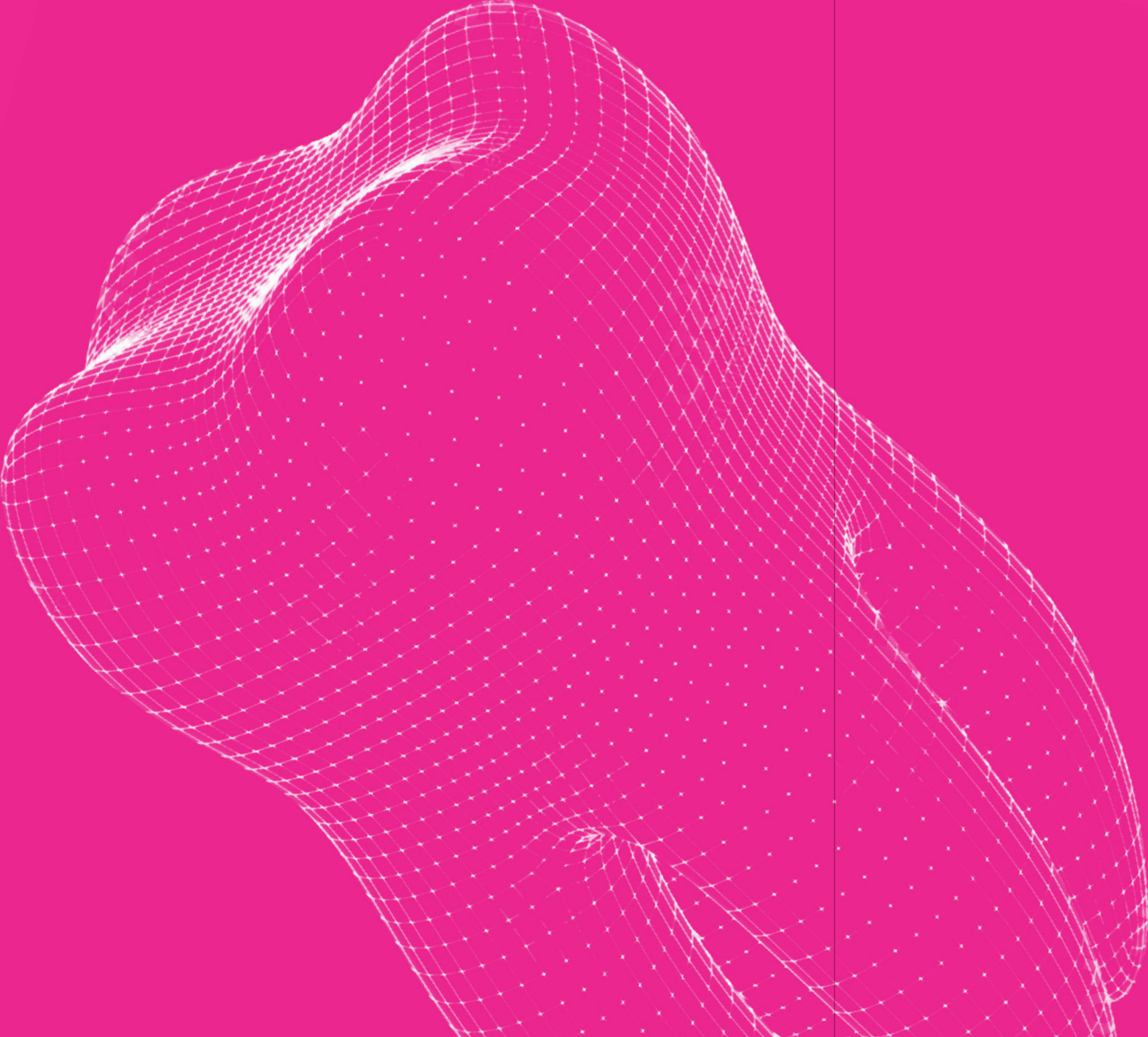

An abstract sculpture of a human head and neck, rendered in a wireframe style. The sculpture is composed of a dense grid of white lines and dots, creating a mesh-like structure. It is positioned on the left side of the page, with the head tilted slightly to the right. The background is a solid dark blue color.

**digital.**

# Digital

GUIDED AXIS  
Ø3.3-Ø4.2 L8-13 KIT

KTG-0001



## TISSUE PUNCH



TKG-0001  
Guided Tissue  
Punch Ø3.3



TKG-0002  
Guided Tissue  
Punch Ø3.75



TKG-0003  
Guided Tissue  
Punch Ø4.2

## BONE MILL & STARTER



TDG-0003  
Guided Bone  
Mill Ø3.3



TDG-0004  
Guided Bone  
Mill Ø3.75



TDG-0005  
Guided Bone  
Mill Ø4.2



TDG-0002  
Guided Starter drill

## DRILLS



TDG-0006  
Guided Pilot Drill  
Ø2.4 L8



TDG-0007  
Guided Pilot Drill  
Ø2.4 L10



TDG-0008  
Guided Pilot Drill  
Ø2.4 L11.5



TDG-0009  
Guided Pilot Drill  
Ø2.4 L13



TDG-0010  
Guided Drill for  
Ø3.3 L8 Implant



TDG-0011  
Guided Drill for  
Ø3.3 L10 Implant



TDG-0012  
Guided Drill for  
Ø3.3 L11.5 Implant



TDG-0013  
Guided Drill for  
Ø3.3 L13 Implant



TDG-0014  
Guided Drill for  
Ø3.75 L8 Implant



TDG-0015  
Guided Drill for  
Ø3.75 L10 Implant



TDG-0016  
Guided Drill for  
Ø3.75 L11.5 Implant



TDG-0017  
Guided Drill for  
Ø3.75 L13 Implant



TDG-0018  
Guided Drill for  
Ø4.2 L8 Implant



TDG-0019  
Guided Drill for  
Ø4.2 L10 Implant



TDG-0020  
Guided Drill for  
Ø4.2 L11.5 Implant



TDG-0021  
Guided Drill for  
Ø4.2 L13 Implant

TOOLS



TD1-0030  
Drill Extender



TDG-0001  
Guided Pilot Drill  
Ø2



TKG-0004  
Guided Lateral  
Fixation Screw



TKG-0005  
Guided Anchoring  
Screw



TKG-0016  
Guided Soft Bone  
Anchoring Screw



TKG-0014  
Guided Implant Driver  
Short



TKG-0006  
Guided Implant Driver  
Long



TKG-0007  
Guided Implant  
Driver Ratchet



TKG-0008  
Guided Implant  
Driver Motor



TKG-0009  
Guided Implant  
Driver Extractor  
Long



TKG-0015  
Guided Implant  
Driver Extractor  
Short



TK1-0006  
Screw Driver  
Long / 1.28mm



TKG-0010  
Guided Drill  
Template



TK1-0015  
Ratchet

GUIDE SLEEVES



TKG-0011

Guided Sleeve  
H4,5 mm



TKG-0012

Guided Sleeve  
Fixation Pin

OPIONAL GUIDE DRILLS



TDG-0022

Guided Drill for  
Ø2.4 L16 Implant



TDG-0023

Guided Drill for  
Ø3.3 L16 Implant



TDG-0024

Guided Drill for  
Ø3.75 L16 Implant



TDG-0025

Guided Drill for  
Ø4.2 L16 Implant

TI BASE

⬡ Anti Rotation   ⬢ Free Rotation



CCA-0001

CAD/CAM  
Ti Base

⬡ Gingival height = 0mm



CCA-0002

CAD/CAM  
Ti Base

⬢ Gingival height = 0mm



CCA-0019

CAD/CAM  
Ti Base

⬡ Gingival height = 1mm



CCA-0023

CAD/CAM  
Ti Base

⬢ Gingival height = 1mm



CCA-0020

CAD/CAM  
Ti Base

⬡ Gingival height = 2mm



CCA-0024

CAD/CAM  
Ti Base

⬢ Gingival height = 2mm



CCA-0021

CAD/CAM  
Ti Base

⬡ Gingival height = 3mm



CCA-0025

CAD/CAM  
Ti Base

⬢ Gingival height = 3mm

# Digital

## TI BASE

### Multi-Unit Level



CCA-0010  
CAD/CAM  
Ti Base for Multi Unit



CCA-0029  
CAD/CAM  
Low Ti Base for Multi Unit

## CAD/CAM TEMPORARY ABUTMENT

### Implant Level

○ Free Rotation



CCA-0026  
CAD/CAM  
○ Temporary Ti Cementing Post



CCA-0027  
CAD/CAM  
○ Temporary Peek Cementing Post

### Multi-Unit Level



CCA-0028  
CAD/CAM  
Temporary Multi Unit  
Peek Cementing Post



CCA-0030  
CAD/CAM  
Temporary Multi Unit  
Ti Cementing Post



CCA-0031  
CAD/CAM  
Low Temporary M.U  
Ti Cementing Post

## TITANIUM BLANK



CCA-0011  
CAD/CAM  
Blank Ø 9mm



CCA-0012  
CAD/CAM  
Blank Ø 12mm



CCA-0013  
CAD/CAM  
Blank for Multi Unit

## SCAN POST



CCA-0014  
CAD/CAM  
Multiuse ScanBody



CCA-0015  
CAD/CAM  
Multiuse ScanBody for Multi Unit

## DIGITAL ANALOG



CCA-0016  
Implant Analog  
for printed model



CCA-0017  
MU Analog  
for printed model

Noga Medical Products Ltd  
P.O.B. 55, Shlomi 2283200, Israel  
Tel: +972(0)4 980 8080  
[www.tagdent.com](http://www.tagdent.com)

T.A.G is ISO 13485 and certified by GMED  
T.A.G Products are CE marked in accordance with Directive 93/42/EEC

DA-CTLGE Rev.7



**Experience the Difference**

T.A.G is ISO 13485 and certified by GMED  
T.A.G Products are CE marked in accordance with Directive 93/42/EEC

DA-CTLGE Rev.7